

SAFETY & MAINTENANCE

We encourage you to seek assistance from a qualified Harmar dealer when necessary. Always adhere to your mobility device's operating instructions. Caution should be used at all times. It is important that you are able to safely operate your mobility device following the degree of incline/decline and chair instructions before attempting ramp use. Always follow the safety requirements from your Harmar Owner's Manual.

OWNER/MAINTENANCE

Daily Upkeep: It is vital to keep all usable surfaces free and clear of dirt, leaves, snow/ice, and any other accumulating debris that can impede the safety of the ramp system. Do not attempt to use your ramp if it is covered in leaves, dirt, debris, snow, and/or ice. It is critical that the non-skid face of the ramp system remains free of buildup to ensure traction remains fully intact. Remember, even non-skid treads can become slippery when wet or in icy conditions. Please use extra caution on a damp/wet surface.

NOTE: Do not use harsh chemicals and/or alkaline chemicals to clean the ramp, as these can damage the chemical compounds of the metal, jeopardizing the structural integrity of the ramp system.

Leaf, Dirt, and Debris Removal: Most debris can be removed by sweeping the ramp's surface with a broom, using a leaf blower, and/or spraying it with a garden hose. For buildup on the surface that cannot be easily swept or rinsed away, damp cloth and/or soft brush with a mild soap & water solution can be used.

Snow and Ice Removal: Snow should be removed with a shovel followed by sweeping the tread until clean & clear. If ice has built up on the ramp, a Magnesium Chloride flake or pellet is recommended to melt the ice. Once the snow/ice has been cleared, it is important to clean the ramp system with a mild soap & water solution.

NOTE: Please refrain from using traditional rock salts as these can damage the welds and chemical compounds of the ramp, jeopardizing the structural integrity of the system.

ROUTINE MAINTENANCE

To maintain the safety of your ramp and its components, it is recommended that the ramp is frequently inspected for damage and to tighten loose fasteners. Should you find that any part of the ramp is damaged or loose, do not attempt to repair this yourself - contact a certified Harmar dealer for assistance.